

ISO  
13485

MDSAP  
MEDICAL DEVICE SINGLE AUDIT PROGRAM




**AEGIS LIFESCIENCES**

PROTECTION THROUGH PERFECTION

**FLOGEL<sup>®</sup>**

FLOWABLE GELATIN HAEMOSTATIC MATRIX KIT 

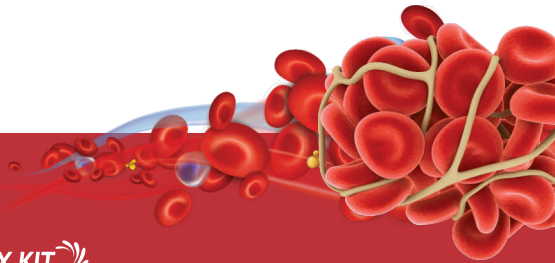

# FLOGEL®

FLOWABLE GELATIN HAEMOSTATIC MATRIX KIT 



*FLOGEL*® Flowable Gelatin Haemostatic Matrix, is made from absorbable gelatin sponge powder, into a flowable form as off white (slight yellow) flowable haemostatic matrix prefilled in a syringe. It is used by applying to a bleeding that occurs in tight and irregular spaces especially at a difficult to reach surgical site.

## KEY FEATURES

- Maintains uniform viscosity from beginning to end of the application after mixing with saline/thrombin
- Ready to use in 30 seconds once the sterile saline/thrombin is uniformly mixed
- Product after constitution (as directed below) envelops bleeding site to stop bleeding faster
- It has a proven efficacy, safety, efficiency across blood transfusions, major and minor complications and surgical operating time
- Stays in place during active bleeding even after mixing with saline/thrombin, so it delivers faster haemostasis where its intended
- It is easy to use and can be applied locally at the target site of bleeding.


An empty syringe for mixing sterile solution with matrix

*FLOGEL*<sup>®</sup> Flowable Gelatin Haemostatic Matrix

A connector luer

A liquid transfer cup

An applicator tip (14 cm)

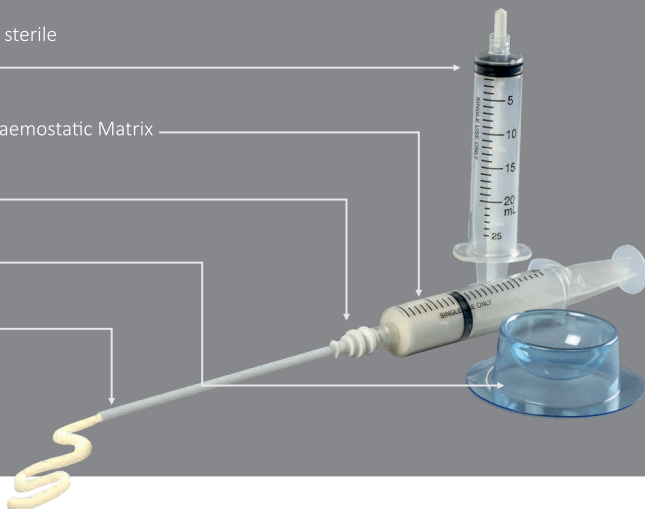

## HAEMOSTATIC MATRIX PREPARATION



Step. 1



Step. 2



Step. 3



Step. 4

**Step 1** - Take sterile saline/thrombin. Transfer sterile solution into the sterile liquid transfer cup. Draw up the sterile solution from the liquid transfer cup into empty syringe.

**Step 2** - Connect the syringes – Remove cap from *FLOGEL*<sup>®</sup> Matrix prefilled syringe and attach this syringe to the sterile syringe carrying sterile solution, through luer connector.

**Step 3** - Mix the contents of syringes – Begin mixing both by pushing the sterile solution into the *FLOGEL*<sup>®</sup> Matrix prefilled syringe. Continue exchanging the solution between syringes until all components are thoroughly mixed and the consistency is even.

**Step 4** - Attach the applicator tip for product to be ready for clinical use.

## ORDERING INFORMATION

Product ID	Product Name	Volume	No. of Pieces Per Box
FG-005	FLOWABLE GELATIN HAEMOSTATIC MATRIX KIT	5ml	1
FG-006	FLOWABLE GELATIN HAEMOSTATIC MATRIX KIT	6ml	1
FG-008	FLOWABLE GELATIN HAEMOSTATIC MATRIX KIT	8ml	1
FG-010	FLOWABLE GELATIN HAEMOSTATIC MATRIX KIT	10ml	1

Do not Inject **FLOGEL**<sup>®</sup> Haemostatic Matrix into blood vessels.

For complete product details, Contraindications, Warnings, Precautions etc. please refer Instructions For Use (IFU).

### AEGIS LIFESCIENCES PVT. LTD.

215/216, Mahagujarat Estate, Gam : Moraiya,  
Tal. Sanand, Dist.: Ahmedabad-382213.

WC3P+RM Changodar, Gujarat, India

+91 9512503131 +91 9512603131

marketing1@aegis-lifesciences.com  
aegis.lifesciences@gmail.com



BROCHURE



WEBSITE

[www.aegis-lifesciences.com](http://www.aegis-lifesciences.com)