



AEGIS LIFESCIENCES

PROTECTION THROUGH PERFECTION

SURGISPON[®] Anal Tampon

Absorbable Haemostatic Gelatin Sponge for Anorectal Use



- Quick haemostasis in anorectal surgeries
- Best suitable for post operative haemostasis
- Painless and easy tampon discharge
- High blood absorption capacity
- SURGISPON[®] Tampon is provided with an opening into which a flatus drain can be introduced.

SURGISPON[®] Anal Tampon

Advantages and Benefits

- Reduces post-operative pains more effectively than other known forms of anal tampons
- High absorption capacity-approximately 45 times of its own weight
- Anal tampon gelatin sponge liquefies rapidly and is spontaneously expelled within 1 to 2 days
- Used as carrier for drugs such as antibiotics, thrombin
- The gelatin sponge application can reduce up to 4 days of hospital stay for rectal patient
- Significantly increases haemostasis without pain and discomfort
- The immediate insertion of an absorbable haemostatic gelatin sponge can reduce the frequency of rectal bleeding.



Features

- Non-toxic
- Non allergenic
- Non-immunogenic
- Non-pyrogenic
- Biocompatible
- It has a water in-soluble property
- Used dry or soaked with physiological solution
- Well adhering ability to the bleeding site
- Implantable and completely resorbable
- Does not stick to instruments.

SURGISPON[®] is manufactured from highly purified & pH neutral pharmaceutical gelatin granules.

SURGISPON[®] is gamma sterilised and single use absorbable haemostatic gelatin sponge.

SURGISPON[®] Anal Tampon is used in haemorrhoid operations and other surgical procedures in the anal, vaginal and rectal regions.

Painless and easy tampon discharge. The SURGISPON[®] is easily expelled at first bowel movement, requiring no additional nursing intervention.

Where to use :

- Anorectal surgery-suitable to facilitate better post operative haemostasis
- To achieve fast haemostasis in :
 - Haemorrhoidectomies
 - Anal fistula and fissures
 - Curettage and Hysterectomies
 - Surgery for Rectal cancer, Rectal prolapse Colorectal surgery, transrectal prostate needle biopsy, for haemostasis
 - Vaginal application.

How to use :

For Rectal Applications

In Anal and Rectal surgery, introduce a dry anal tampon with the help of a proctoscope at the end of the surgical procedure.

- Place dry gauze in the rima natis
- The buttocks must be held together by means of a T bandage
- Bacterial invasion ensures the rapid dissolution of the sponge which softens and is spontaneously removed by defecation
- The SURGISPON[®] Tampon is provided with an opening into which a flatus drain can be introduced.



After performing excision of hemorrhoidal piles, the operation is terminated by placing haemostatic absorbable gelatin sponge tampon in the anal canal.

For Vaginal Applications

Shape SURGISPON[®] Tampon to fit speculum.
Insert, by means of a speculum a dry or soaked SURGISPON[®] Tampon.

SURGISPON[®] Tampon may be used in combination with antibiotics, thrombin, chemotherapeutics without reduction of the haemostatic effect. Carefully read applicable instructions for use (IFU) before application.

Biological Safety & Performance Evaluation¹ has been performed on SURGISPON[®] according to ISO 10993 and USP standards. A summary of these studies can be obtained on demand.

Product ID	Product Name	Dimensions (mm)	No. of Pieces Per Box
SSP-8030	Anal Tampon	80xø30	06

 A-504, Swati Trinity, Nr. Shantipura Circle, SP Ring Road, Ahmedabad, Gujarat-380058, India.

 215/216, Mahagujarat Industrial Estate, Ahmedabad, Gujarat-382213, India.

 +91-951 290 3131  +91-951 288 3131

 marketing1@aegis-lifesciences.com
aegis.lifesciences@gmail.com

 www.aegis-lifesciences.com



Important: Please refer Instructions for Use (IFU) for complete instructions, contraindications, warnings and precautions.

AL/BRC/SSPAT/01/R5 | Date: 11/2025